

NOCOSPRAY

Qaabka loo isticmaalo



Si taxaadar leh uu akhri buuga isticmaalka oo Dhaan oo la socda ROBODKA NOCOSPRAY!



1 Isticmaal gaacmo gashii iyo muraayada indhaha ilaaliso markaad la tacaalayso dhamman kiimikooyinka loo isticmalo robotka.




2 Hubi in dhaladu ay ku jirto xooga ku filan kiimikoo ah. Calaamadee heerka kaa hor nidaamba oo hubi ka dibna.



3 Ku marooji sanka buufinta dhalada. Fuurka meel ficaan dhiig.



4 Guji birta buufinta dhalada ku jirta meel ku taal Robotka Nocospray. Iska hubi saanka tubada iin amman yahay.

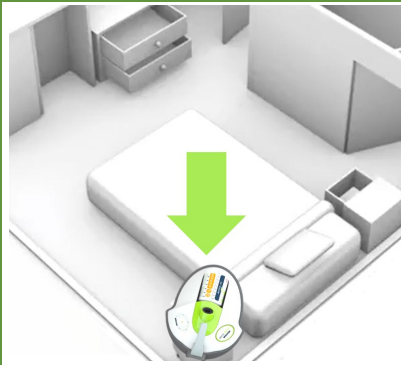

5 Dhiig Robotka Nocospray meel adag. Dooro gees si aad saahmeyn wanaagsan ugu yeelato.



Nocolyse One Shot: Muuga qoolka (m³) **x3**

Nocolyse Neutral: Muuga qoolka (m³) **x1**

6 Cabiir qiiyasta qolka saxda ah. Gobtu waxay ku xirantahay weyniinka qolka iyo nooca kiimikada ah ee la isticmaalo.



7 Riix bilawga dhexe ee garaaca lambarka. Robotka waxu dhawaaqaa 15 aqiqo wuxuuna si tos ah u bilaabmaa.



Nocolyse One Shot: **2 saacadaha** wakhtiga xidhiidhka

Nocolyse Neutral: **30 daqiiqado** wakhtiga xidhiidhka

8 Qolku waa in la xidhaa/la qufulaa inta lagu jiro faafinta wakhtiga xidhiidhkaa. Albaabka ku dheji boodh ama warqaad ku qoran tahay digniin ku saabsan inaan qolka la Geeli kariin.



9 Dabool dhalada. Daboolka dib ugu celi. Biyo raaci tubada buufinta iyo tubada biyaha hoostoda. Guji birta oo ku celi robotka booskisa.

Ka bax qolka !